

Order Information:

SA-DA 75 SmartAnchor Dual System 7,5 mm. 5/box
SA-DI 75 SmartAnchor Dual System Impactor 7,5 mm.

Anchors are Gamma Sterilized

CE 0120

Warranty:

MD Supply and CPP warrants, that reasonable care was exercised during the production of this product. Since MD Supply and CPP have no control over the conditions of use, patient selection, surgical procedure, post-surgical stresses or handling of the product after it leaves our possession, MD Supply and CPP does not warrant either a good effect or an ill effect after its use.

Produced for MD Supply by:
CPP-Clinical Plastic Products SA
Allée du Quartz 9
2300 La Chaux-de-Fonds
Switzerland

For additional information
contact your local representative.

Patent Pending

Distributed by:

CJ Medical
12 Long Wall . Haddenham . Buckinghamshire HP17 8DL . UK
Phone and Fax: +44 (1844) 299501
E-mail: info@cjmedical.com . Website: www.cjmedical.com

Application guide:

① Inspect the packaging for pinholes.
Open the peel-pack and take out the applicator.

② Turn the safety pin to vertical position.

③ Remove the safety pin.

④ Introduce the suture of choice through the anchor tunnels.

⑤ Lock the suture and anchors by twisting the suture around the self-locking buttons on the Impactor.


⑥ Release the Impactor by opening the snap lock.

Precaution:

The impactor has to be inspected for damage before use.

The choice of suture is the responsibility of the surgeon.

Anchors, for single use only
Impactor is reusable.

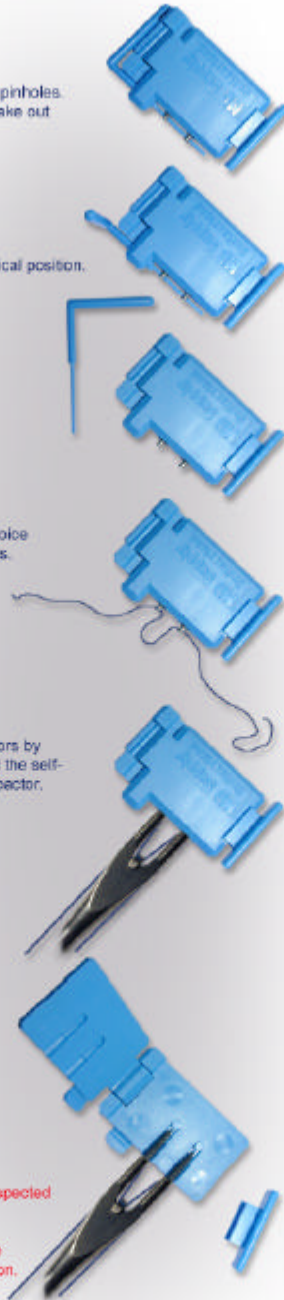

MD
Supply
Switzerland

