


# A.M.I.® Soft Anal Band System

Product Group Coloproctology | Issue 11/2016

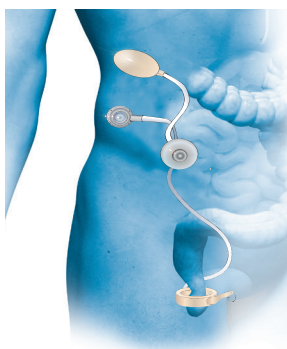

## Long-term implant for the treatment of faecal incontinence in adults

- All parts are implanted subcutaneously
- The positioning of components is patient-specific
- A quick glance shows if the sphincter is closed
- The closing pressure is easily adjustable at any time





# A.M.I.® Soft Anal Band System

The A.M.I. Soft Anal Band is a long-term implant for the treatment of faecal incontinence in adults.

Faecal incontinence affects both women and men of all ages, and can dramatically impair their quality of life. This may lead to severe mental disorders, which then may result in isolation and depression. In addition, many of the sufferers are elderly people requiring home care, which is made all the more difficult by the incontinence.

## Features:

- Easy manual operation gives patients total control of the sphincter function.
- Subcutaneous implantation means no body cavities need be opened.
- System operation and positioning of parts can be adjusted to suit individual patient requirements.
- A quick glance at the activator lying under the skin shows whether the artificial sphincter is open or closed.

Order Code	Product	Technical Details
AAB 300	<p><i>A.M.I. Soft Anal Band System</i> Consisting of:</p> <ul style="list-style-type: none"> <li>– A.M.I. Soft Anal Band Ring (cuff) in two different sizes M-L plus one extension part</li> <li>– A.M.I. Soft Anal Band Valve</li> <li>– A.M.I. Soft Anal Band Activator M-L</li> <li>– A.M.I. Safe Puncture Port Low Profile</li> <li>– A.M.I. Soft Anal Band Scale</li> <li>– A.M.I. Port Needle</li> </ul>	<p>1 unit each</p> <p>1 unit 1 unit each 1 unit 1 unit 2 units</p> <p>All units delivered sterile</p>
	<p><i>A.M.I. Soft Anal Band Ring</i> After measuring the required cuff size, the cuff is then placed inside the surgically created circular pocket around the non-functioning anal sphincter.</p>	The adjustable, artificial sphincter (the cuff) encircles the non-functioning anal sphincter.
	<p><i>A.M.I. Soft Anal Band Valve</i> Small and stable with a rounded top. Once activated, liquid flows from the cuff back to the activator allowing the artificial sphincter to open.</p>	The valve is activated by applying finger-pressure on the skin above the valve.
	<p><i>A.M.I. Soft Anal Band Activator</i> Strong and durable silicone balloon. Applying pressure (with the palm) on the skin above the activator causes liquid to flow into the cuff. This closes the artificial sphincter, leaving the patient continent.</p>	The status of the artificial sphincter – OPEN or CLOSED – can be checked at any time. Protruding skin above the activator indicates an open sphincter. Flat skin indicates that the sphincter is closed.
	<p><i>A.M.I. Safe Puncture Port Low Profile</i> Used by surgeon for patient-specific adjustment of the liquid volume in the system.</p>	Made of titanium Protection sleeve prevents tube kinking.

## A.M.I. Headquarters:

A.M.I. GmbH  
Im Letten 1  
6800 Feldkirch . Austria  
t +43 5522 90505-0  
f +43 5522 90505-4006  
e info@ami.at  
[www.ami.at](http://www.ami.at)



**A.M.I.**®