


## i-Stitch and i-Stitch up

Application Manual


IST1010 / IST1040

Available i-Stitch loading units:


IST1021, IST1051, IST1011, IST1031

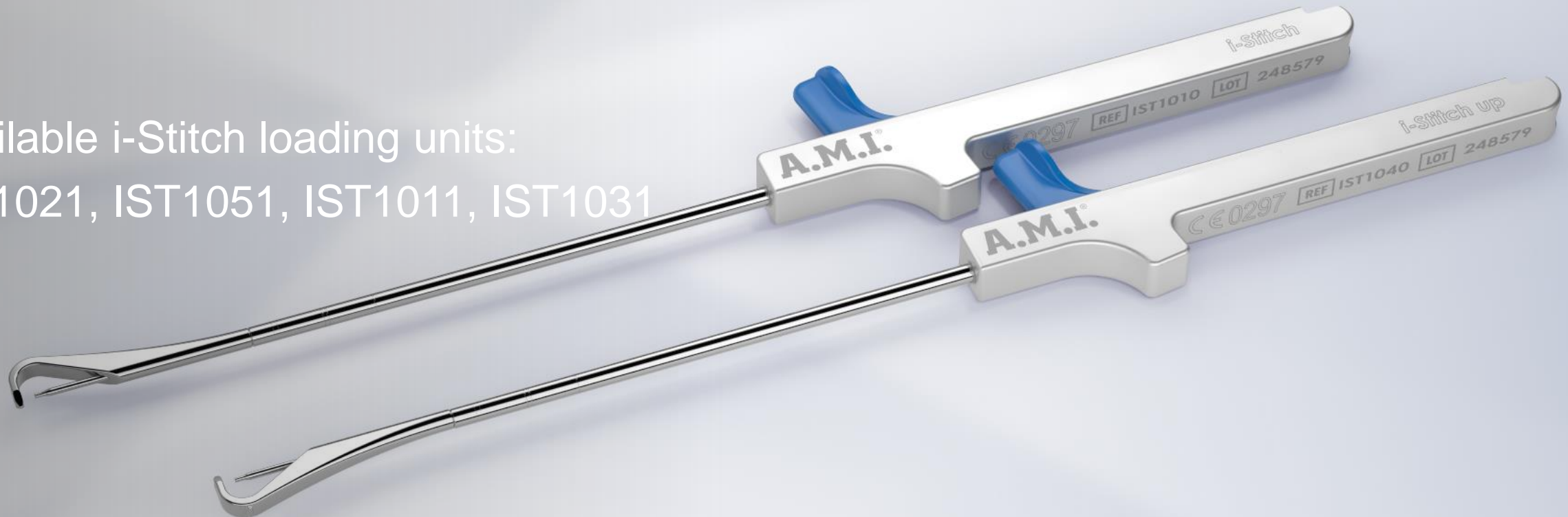

# i-Stitch, i-Stitch up and i-Stitch loading unit

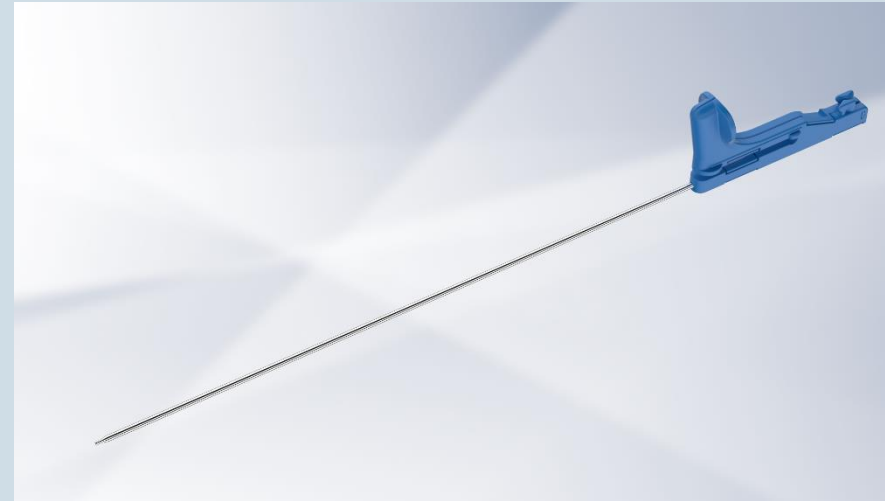

## Secure suture

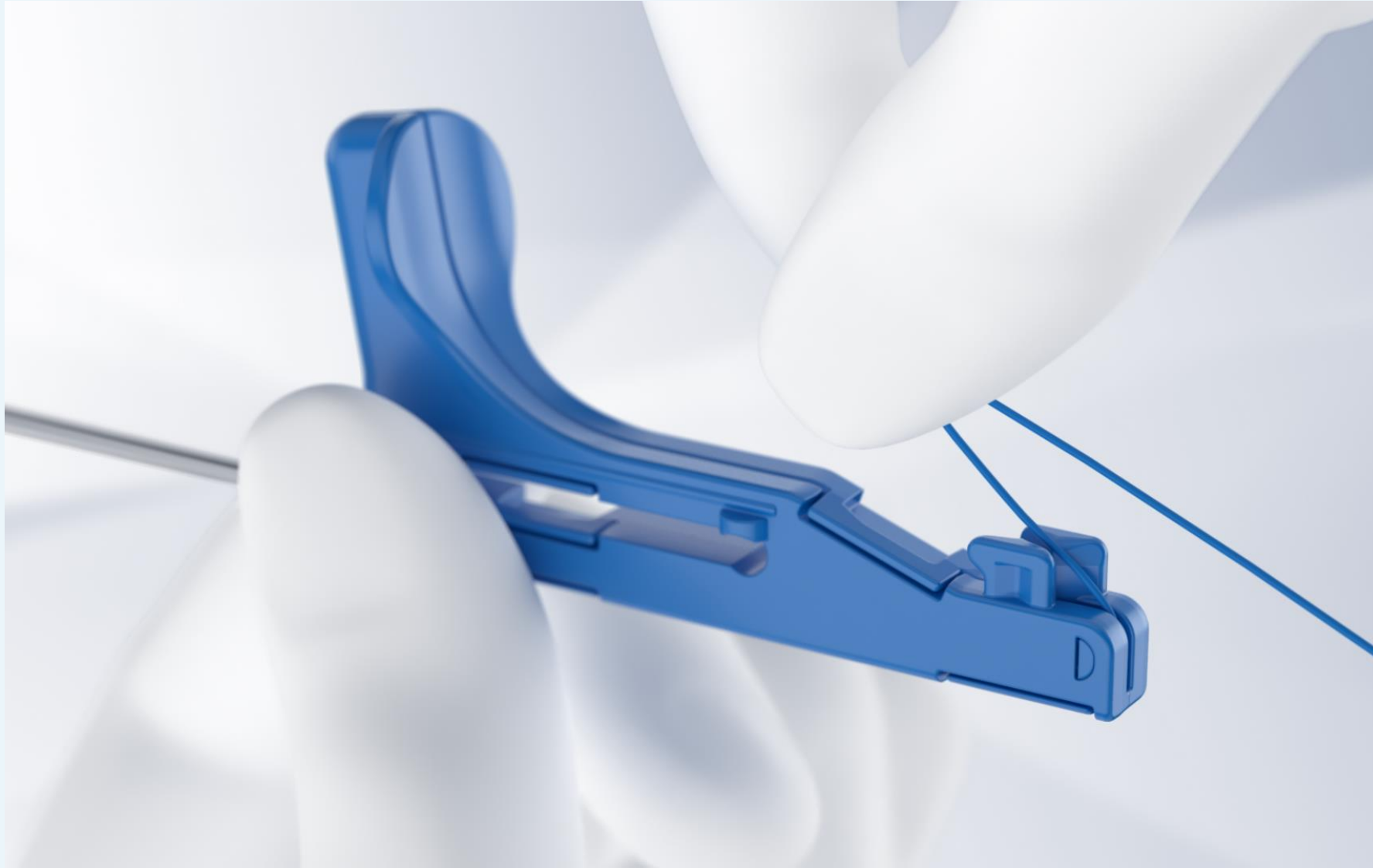

- Take the i-Stitch loading unit.
- Gently pull the suture backwards, so that the blunt suture tip is positioned directly at the end of the cannulated needle.
- Secure the suture with the suture clamp.


## Insert i-Stitch loading unit

- Insert the i-Stitch loading unit into the i-Stitch guiding slot.
- Make sure that the loading unit fits correctly in the guiding slot.
- Insert the loading unit to approx. 1 cm before the end of the guiding slot (stop position), you will feel a slight resistance.




## Place index finger

- Place your index finger at the desired target position.
- Guide the blunt instrument tip along your index finger to the desired position.
- Depth markings on the tip serve as a guide to determine insertion depth of the tip.

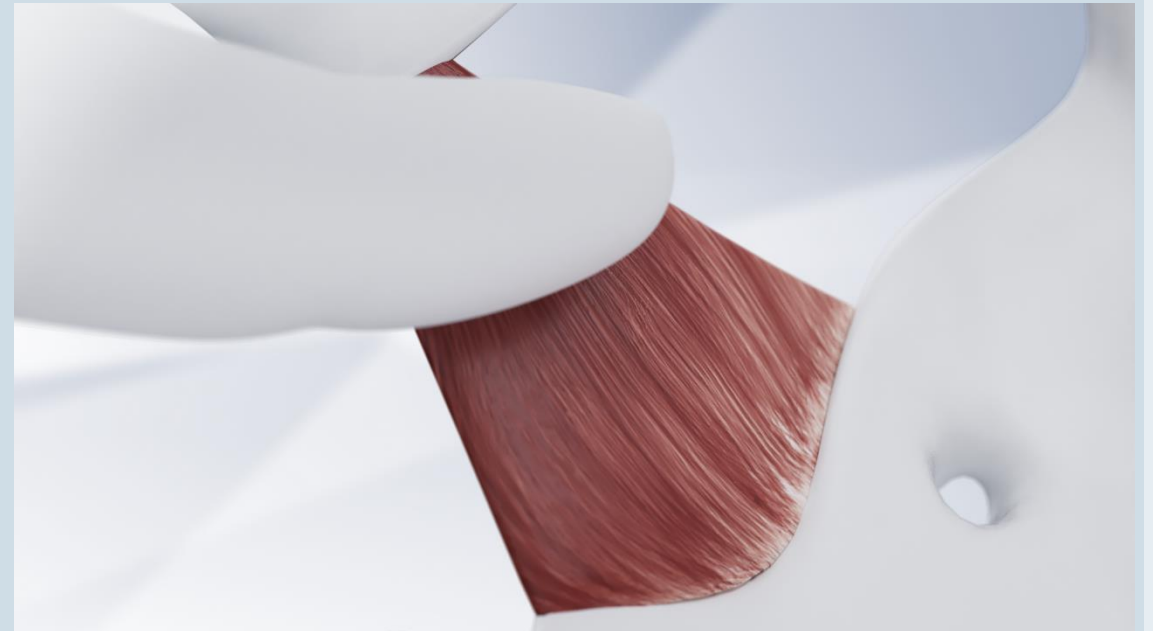


## Push tip into ligament

- Push the tip of the instrument into the sacrospinous ligament using your index finger (the reference marking indicates the correct position).
- Attention: press the index finger on the tip until you remove the entire i-Stitch instrument.

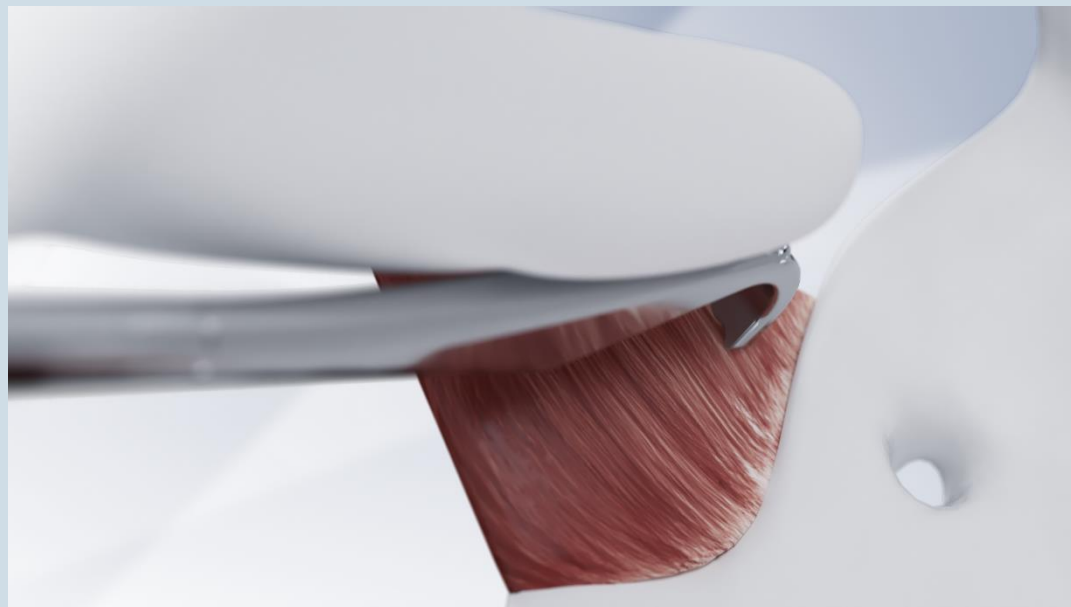


## Push loading unit forward

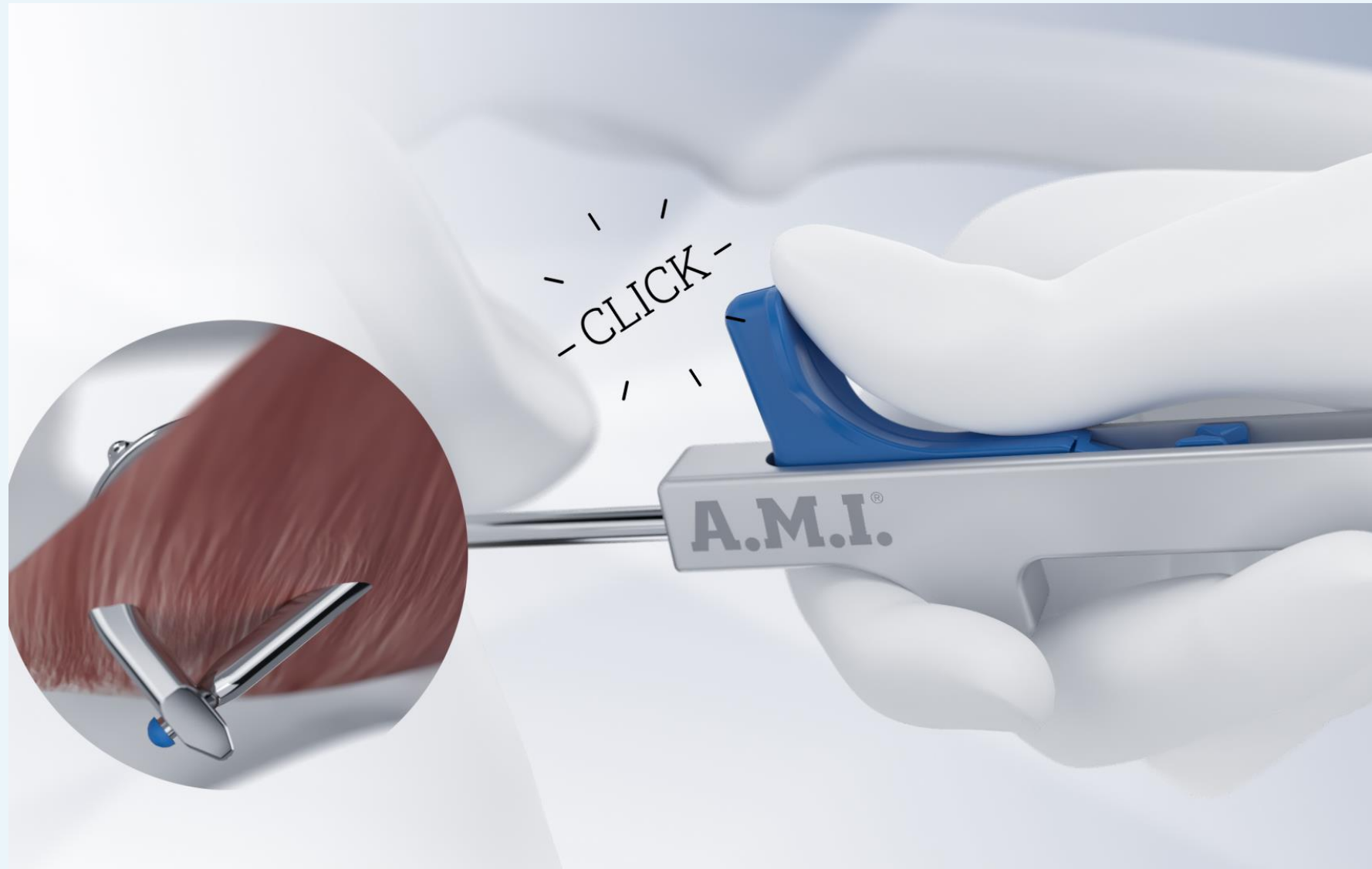

- Push the loading unit forward with your thumb to release the instrument.
- An audible click can be heard.
- The blunt suture tip penetrates the tissue and passes through.
- The blunt suture tip is secured in the i-Stitch's tip.

## Release suture from suture clamp



- Before removing the loading unit, release the suture from the suture clamp. It will no longer be secured.
- Withdraw the loading unit completely from the i-Stitch/i-Stitch up.

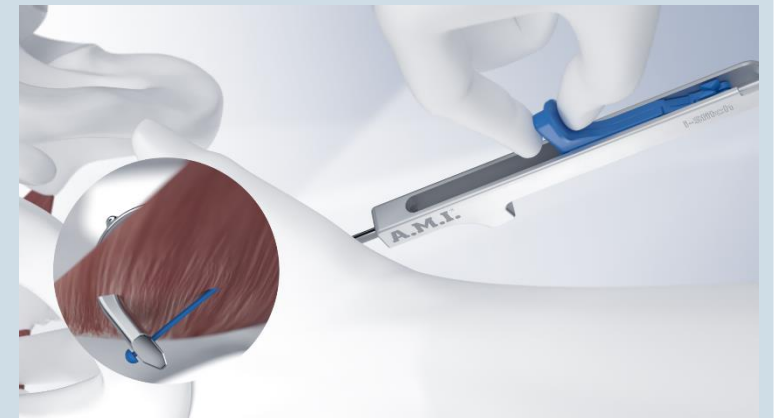




## Lift up tip

- Remove your index finger from the tip of the i-Stitch instrument.
- Lift the tip of the instrument slightly and remove the instrument.
- The suture will remain firmly in place.

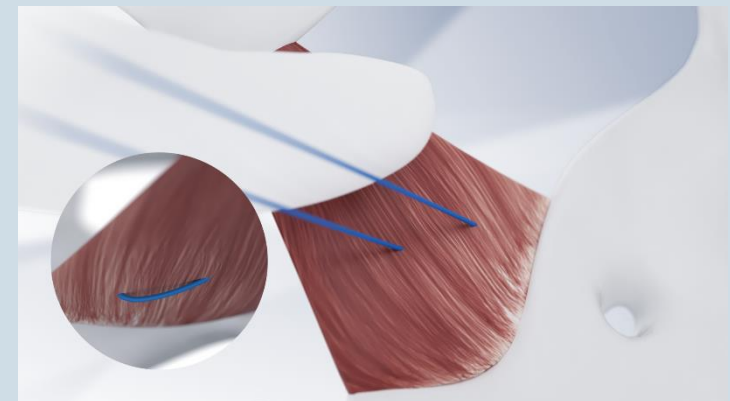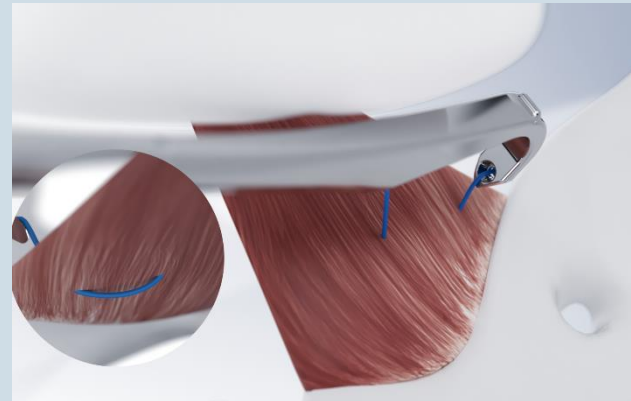

## Remove i-Stitch or i-Stitch up


- Remove the i-Stitch or i-Stitch up instrument. Grasp the two endings of the suture.
- Make sure that the suture is tight and correctly attached.

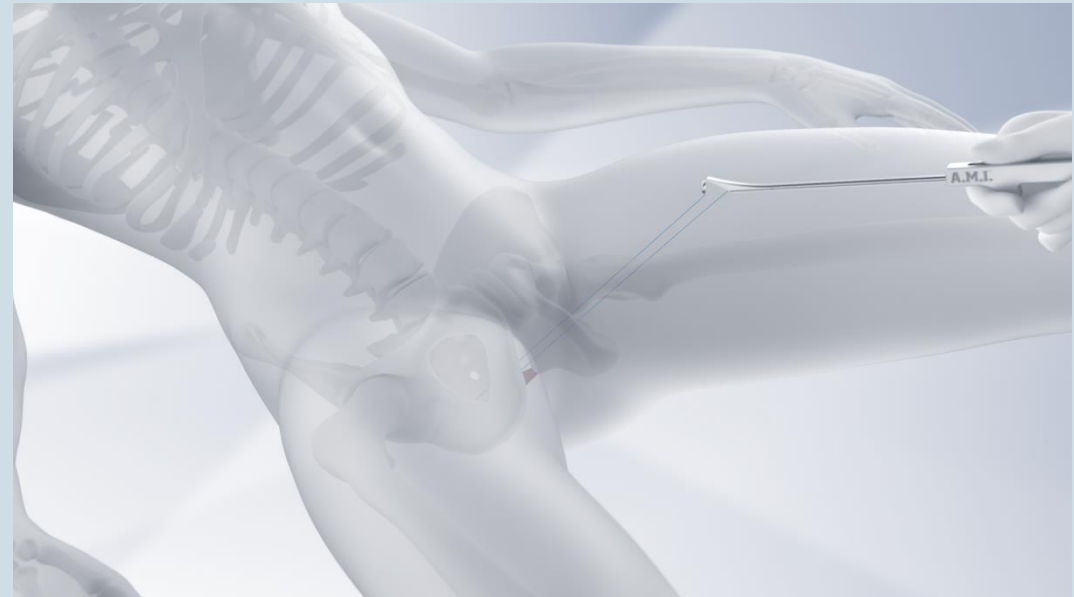

**A.M.I.<sup>®</sup>**

**Thank you for your  
attention!**

Please have a look at our animation.  
[i-Stitch animated video](#)

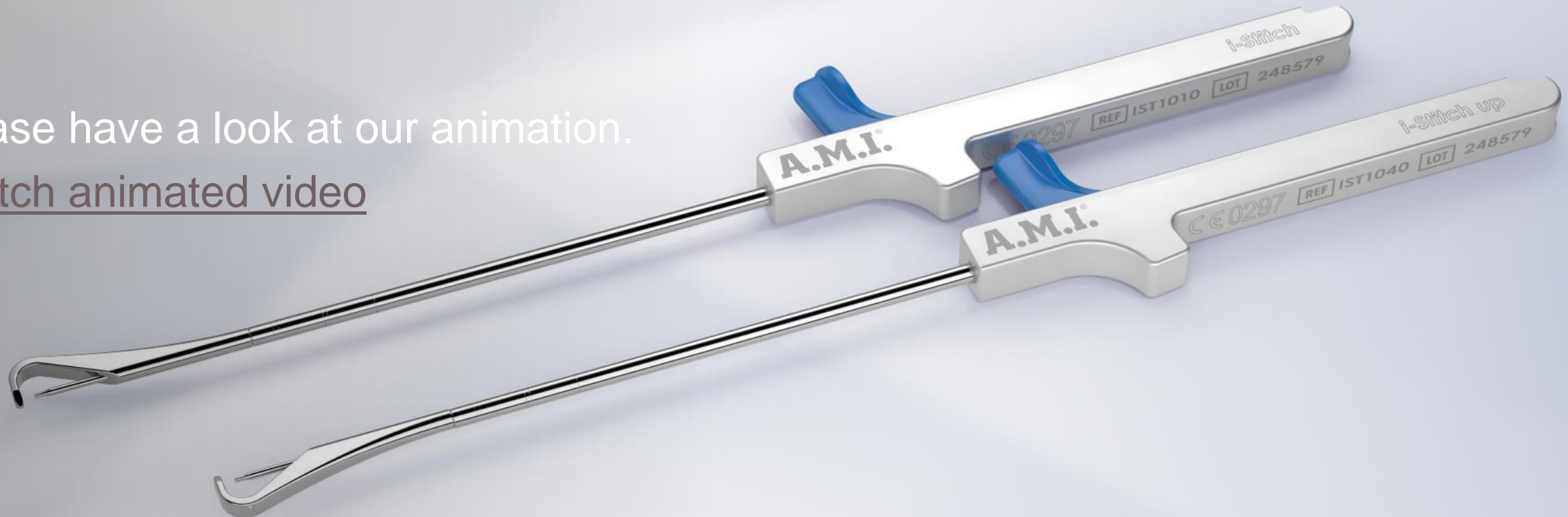

Feldkirch, July 2021